Store at 2 - 8 °C

Configuration

REF	HBS02
VOL	1 mL
Hemoglobin Calibrator	1 x 1 mL
Instrument	Universel

Intended use

The Hemoglobin Calibrator is intended for the determination of hemoglobin concentration in human blood. The calibrator is used for calibration in manual and automated procedures.

For *in vitro* diagnostic use only. For professional use only.

Reagent composition

Hemoglobin Calibrator	Animal blood, stabilized (15 g/dL)
-----------------------	------------------------------------

Precautions

- The calibrator doesn't contain components of human origin. However, it is recommended to handle it with due care.
- All body fluid samples should be considered potentially infectious materials and the appropriate precautions should be taken. Wear personal protective equipment such as gloves, safety glasses, lab coats or aprons when working with possible biohazard contaminants.
- Use Good Laboratory Practices (GLP) when handling this product.
- Please refer to the MSDS, available on our website, for further information.

Preparation

The calibrator is ready for use.

Storage, stability and disposal

All the components of the kit are stable up to the date of expiration as specified on the label, when stored tightly closed, protected from light and contaminations prevented during their use. Storage temperature for this kit is 2 - 8 °C.

Do not use the product if deterioration or contamination is suspected or beyond the expiration date or open container stability period. Dispose unused or deteriorated product and waste in compliance with local regulations.

Procedure

The calibrator must be used according to the test instructions of the hemoglobin assay.

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